medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

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power added compression™
(pac band™)

CIRCAID

circaid® power added compression™
(pac band™)
inelastic compression system

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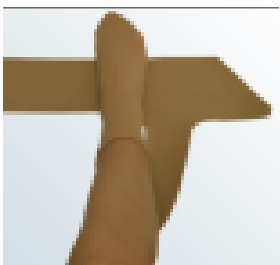
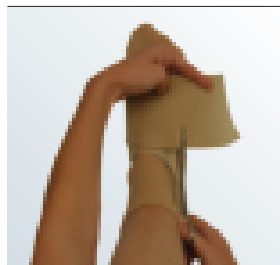
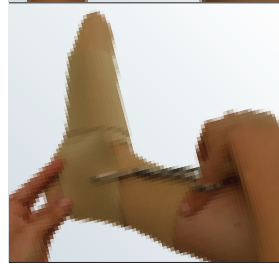
3.1

3.2

circaid® power added compression™ (pac band™)

Purpose

The garment is a single patient, reusable, non-invasive, non-sterile device designed to provide compression to the foot and ankle region for patients with venous and lymphatic disorders. It is intended to be used in conjunction with the circaid compression anklet, however can also be used as a standalone product as well.

Indications

- Lymphedema
- Other forms of edema
 - Venous
 - Post traumatic
 - Post surgical
 - Lipedema
- Varicose veins (all types)
- Chronic venous insufficiency

(CVI)

- Venous stasis disease
- Venous valvular insufficiency (VVI)
- Venous insufficiency
- Post thrombotic syndrome
- Venous ulcer (stasis ulcer)
- Angiodysplasia
- Post sclerotherapy
- Thrombosis prevention
- Venous eczema
- Lipodermatosclerosis

Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the leg
- Intolerance to compression material

Absolute contraindication

A condition that completely

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Italia S.r.l.
Via Guido Rossa 28
40033 Casalecchio di Reno-Bologna
Italy
T +39 051 613 24 84
F +39 051 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
infos@medi-france.com
www.medi.nl

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T +47 52 84 45 00
F +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Orient M.E.A. FZC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
T +971 4 2557344
F +971 4 2557399
md@taurusgroup.net

medi Polska Sp. z o. o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T+48 32 230 60 21
F+48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

circaid® power added compression™ (pac band™)

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